


ANTIGEN TESTING









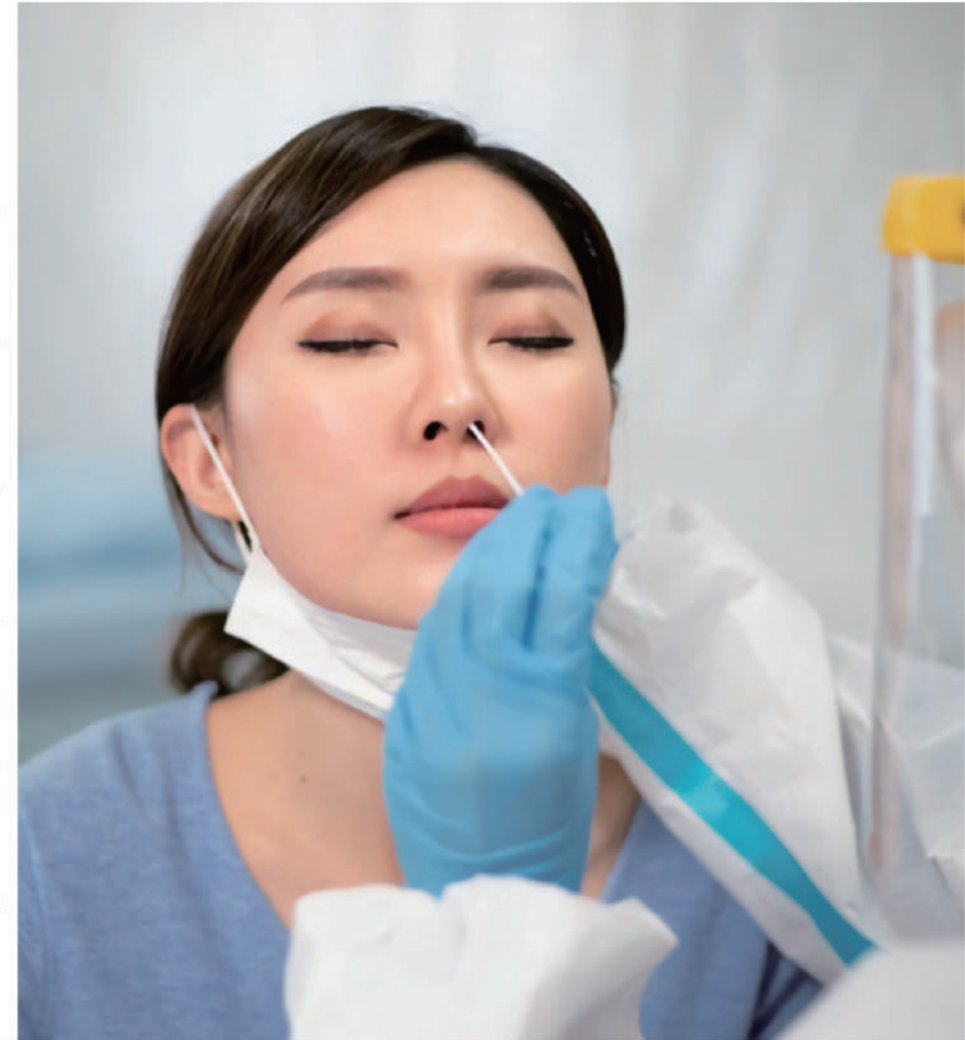


- Easy To Use
- Easy To Carry
- High Sensitivity And Specificity
- Fast Results At 10 Minutes
- Able To Detect Early Infection

White Listed In

- | | | | |
|---|----------------|---|---|
| 1 | EU |  EUROPEAN COMMISSION
DIRECTORATE-GENERAL FOR HEALTH AND FOOD SAFETY
Public health, country knowledge, crisis management
Health Security | |
| 2 | Germany |  Federal Institute
for Drugs
and Medical Devices | 9 |
| 3 | France |  MINISTÈRE
DE L'EUROPE
ET DES AFFAIRES
ÉTRANGÈRES | 10 |
| 4 | Belgium |  fagg
federal agency for
general aviation
general aviation | 11 |
| 5 | Austria |  Austrian
Federal Office for
Safety in Health Care
BASG | 12 |
| 6 | Czech Republic |  MINISTERSTVO ZDRAVOTNICTVÍ
Palackého náměstí 375/4, 128 01 Praha 2 | 13 |
| 7 | Slovakia |  PUBLIC HEALTH AUTHORITY
OF THE SLOVAK REPUBLIC | 14 |
| 8 | Slovenia |  REPUBLIC OF SLOVENIA
GOV.SI | 15 |
| | | | Poland |
| | | |  gov.pl |
| | | | Bulgaria |
| | | |  Ministry of Health |
| | | | Malaysia |
| | | |  Medical Device
AUTHORITY
MALAYSIA |
| | | | Chile |
| | | |  Ministerio de Salud |
| | | | Ecuador |
| | | |  U.S. Embassy & Consulate
in Ecuador |
| | | | Lithuania |
| | | |  VALSTYBINĖ AKREDITAVIMO SVEIKATOS PRIEŽIŪROS VEIKLAI TARNYBA
PRIE SVEIKATOS APSAUGOS MINISTERIJOS |
| | | | Thailand |
| | | |  ทั่วประเทศ
ให้บริการประชาชนคนไทยให้ปลอดภัย |



Self Testing In

- | | | | |
|------------------|---|-------------|--|
| 1 CE1434 | 
CERTIFICATE | 6 Lithuania | 
VALSTYBINĖ AKREDITAVIMO SVETIKATOS PRIŽIŪROS VEIKLAI TARNYBA
PRIE SVETIKATOS APSAUGOS MINISTERIJOS |
| 2 Germany | 
Federal Institute
for Drugs
and Medical Devices | 7 Malaysia | 
Medical Device
Authority
MALAYSIA |
| 3 France | 
MINISTÈRE
DE L'EUROPE
ET DES AFFAIRES
ÉTRANGÈRES | 8 Thailand | 
ไทยรับราชการประเมินเทคโนโลยีทางการแพทย์ |
| 4 Czech Republic | 
MINISTERSTVO ZDRAVOTNICTVÍ
Palackého náměstí 375/4, 128 01 Praha 2 | 9 Bulgaria | 
Ministry of Health |
| 5 Austria | 
Austrian
Federal Office for
Safety in Health Care
BASG | | |



Validated In

Germany

The test has been evaluated and approved by a reputable laboratory from Germany:
Clinical Study Results (>100 positive samples; > 100 negative samples):

1. Analytical Results with correlation to Ct-values of the positive samples:

Ct-value	No. of Samples	No. of true positive Rapid Test Samples	Sensitivity of SARS-CoV-2 Antigen Rapid Test
≤30	82	82	100%
≤32	94	92	97.9%
≤34	102	98	96.1%
≤36	109	103	94.5%

2. Analytical Results with correlation to Ct-values of the negative samples:

No. of Samples	No. of true negative Rapid Test Samples	Sensitivity of SARS-CoV-2 Antigen Rapid Test
82	82	100%

- 1 France:**
SPIRAL Evaluation with good results: Sensitivity 97.1%, Specificity 100%
- 2 Malaysia:**
IMR(Institute for Medical Research) Evaluation with good results: Sensitivity 96.0%, Specificity 100%
- 3 Thailand:**
Evaluation with good results: Sensitivity 100%, Specificity 100%

Product Details

1 TEST

Materials Required and Provided:

- Test Cassette
- Extraction Kit
- Sterile Swabs
- Package Insert
- Tube Holder
- Biohazard Waste Bag
- Qualification Certificate

Materials Required but not Provided:

The timer and Disinfection products, such as hand sanitizer, rubbing alcohol, soap, etc



Sejoy SARS-CoV-2 Antigen Rapid Test Cassette-Export Packing Cartons

Measurement of Gift Box	Measurement of Carton	Weight of Carton	Packing Quantity
122*65*15mm	50.8*41*41.5cm	GW:14.5 KGS NW:13.5 KGS	600pcs/carton

5 TESTS

Materials Required and Provided:

- 5*Test Cassette
- 5*Extraction Kit
- 5*Sterile Swabs
- 1*Package Insert
- 1*Tube Holder
- 5*Biohazard Waste Bag
- 1*Qualification Certificate

Materials Required but not Provided:

The timer and Disinfection products, such as hand sanitizer, rubbing alcohol, soap, etc



Sejoy SARS-CoV-2 Antigen Rapid Test Cassette-Export Packing Cartons

Measurement of Gift Box	Measurement of Carton	Weight of Carton	Packing Quantity
165*75*37mm	61.7*39.5*35cm	GW:12.1 KGS NW:11.1 KGS	720pcs/carton



Product Details

25 TESTS

Materials Required and Provided:

- 25*Test Cassette
- 25*Extraction Kit
- 25*Sterile Swabs
- 1*Package Insert
- 1*Tube Holder
- 1*Biohazard Waste Bag
- 1*Qualification Certificate

Materials Required but not Provided:

The timer and Disinfection products, such as hand sanitizer, rubbing alcohol, soap, etc



Sejoy SARS-CoV-2 Antigen Rapid Test Cassette-Export Packing Cartons

Measurement of Gift Box	Measurement of Carton	Weight of Carton	Packing Quantity
200*150*70mm	62*42*37.5cm	GW:10.5 KGS NW:12.5 KGS	1000pcs/carton

Soft Pack



Materials Required and Provided:

- 25*Test Cassette
- 25*Extraction Buffer with Integrated Extraction Tube
- 25*Extraction tube cover
- 25*Sterile Swabs
- 1*Package Insert
- 1*Tube Holder
- 1*Biohazard Waste Bag
- 1*Qualification Certificate

Materials Required but not Provided:

The timer and Disinfection products, such as hand sanitizer, rubbing alcohol, soap, etc

Sejoy SARS-CoV-2 Antigen Rapid Test Cassette-Export Packing Cartons

Measurement of Gift Box	Measurement of Carton	Weight of Carton	Packing Quantity
122*65*15mm	50.8*41*41.5cm	GW:14.5 KGS NW:13.5 KGS	600pcs/carton